PATCH ELECTROSTIMULATION

PARCHES DE ELECTROESTIMULACIÓN / ELECTROSTIMULATION

PATCHES / ELEKTROSTIMULATIE PATCHES

INDICATIONS A CONSERVER / CONSERVAR ESTAS INDICACIONES / INDICATIONS TO KEEP / INDICATIES TE HET HOUDEN.

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EN : Contraindications of electrostimulation: In case of doubt and special health problems, it is recommended to seek the advice of a health professional. Contraindicated in all cases of: People with electronic devices such as pacemakers and intracardiac defibrillators. People suffering from epilepsy. Pregnant women. Place the electrodes on the front of the chest area of people with cardiac arrhythmias. Stimulate the lower limbs in case of venous thrombosis or severe arterial obstruction (ischemia). In case of abdominal or inguinal hernia. Autoimmune diseases. Metabolic diseases. Cardiovascular diseases. Respiratory diseases. Taking medication or food supplements. Addictions. People over 50 years old. In case of fever. In case of open skin wounds and eczema. Serious illnesses, bacterial infections or inflammatory processes. Sun or general burns. Skin intolerances. Recent operations. Thrombosis, thrombophlebitis. Stents that have been active for less than six months. Arteriosclerosis, arterial circulatory disorders. Untreated arterial hypertension. Cardiac arrhythmias. Other cardiovascular diseases. Bleeding disorders (haemophilia). Severe forms of diabetes mellitus. Cancer or tumor diseases. Acute arthritis. Neurological diseases and epilepsy. Progressive muscular dystrophy. Abdominal or inguinal hernias. Undiagnosed back pain. Diseases of the internal organs. The electrodes should never be applied to the neck or face.

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